

# revyve™

ANTIMICROBIAL WOUND GEL SPRAY



WITH COACTIV+®  
TECHNOLOGY

## DESCRIPTION

---

revyve® Antimicrobial Wound Gel is a white, odorless gel that provides a moist wound environment conducive to healing. revyve® Antimicrobial Wound Gel provides preservative properties through an antimicrobial (PHMB) to help inhibit microbial colonization within the gel during shelf storage. Chronic wounds are known to contain non-viable tissue. revyve® Antimicrobial Wound Gel can facilitate debridement through a moist wound environment.

## INDICATIONS FOR USE

---

revyve® Antimicrobial Wound Gel is indicated for the management of ulcers (including diabetic foot ulcers, leg ulcers and pressure ulcers), 1st and 2nd degree burns, partial and full thickness wounds, large surface area wounds, and surgical incisions.

## CONTRAINDICATIONS

---

- revyve® Antimicrobial Wound Gel should not be used if there is a history of allergy to any of the ingredients.
- Do not use in combination with anionic surfactants.
- Do not use on exposed cartilage.
- Do not use or mix with other cleansers, soaps, lotions, or ointments as it may affect efficacy.
- Do not use in the middle or inner ear.

## WARNINGS

---

- For EXTERNAL USE ONLY on wounds.
- Do not use if tamper proof seal is damaged.
- Do not use for infusion or injection.
- Do not ingest.
- Do not use in the eyes.
- Do not contaminate opening.
- Not for use on children.
- Keep out of reach of children.
- Contents under pressure; do not puncture or incinerate.

Although rare, a temporary and mild burning sensation may occur upon application. If the sensation persists, rinse product from wound with sterile saline or cold water and discontinue use.

## INGREDIENTS

---

Purified water, Poloxamer 407, Glycerol, Trisodium citrate, Polyhexanide (0.1% PHMB), Citric acid anhydrous, Disodium EDTA.

The PHMB in the formulation serves as a preservative to prevent microbial growth within the gel during shelf storage.

## INSTRUCTIONS FOR USE

---

Prior to application, cleanse and rinse the wound bed.

Application:

1. Remove the cap from the can, hold spray can 5-10 cm above wound surface
2. Press actuator to apply a generous amount of gel to cover entire wound
3. Apply appropriate secondary dressing per institutional protocol suitable for wound depth and amount of exudate.

revyve® Antimicrobial Wound Gel can be used to sustain wound moisture for the duration of time between dressing changes. Change dressings to follow clinical protocol or as per healthcare provider's instruction. revyve® Antimicrobial Wound Gel is water soluble and can be easily irrigated from the wound at dressing change using cold water or sterile saline.

## STORAGE

---

Store with cap closed at room temperature. Avoid excessive heat. Do not refrigerate or freeze.

Preservation and shelf-life:

For single-patient use. The product has a use life of up to 60 days after the can has been opened. Discard 60 days after opening. Unopened product maintains efficacy until labeled expiration date.

## GLOSSARY

---



Do not use if package is damaged



Store at room temperature



Consult instructions for use



Single patient - multiple use



Caution: container may explode if heated

Rx Only  
Latex Free

Manufactured for:  
Kane Biotech Inc.  
190-100 Innovation Drive  
Winnipeg, MB R3T 6G2

[kanebiotech.com](http://kanebiotech.com)

1-855-682-1560  
[info@kanebiotech.com](mailto:info@kanebiotech.com)

REVVYE® and COACTIV+® are registered trademarks of Kane Biotech Inc.

Made in Canada

Packaged in Sweden

REV-11.05.25